
OMRON

Wrist Blood Pressure Monitor

Model R2 Instruction Manual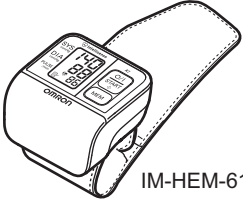

IM-HEM-6113-RU-05-10/2011 5329525-2C

Introduction

Thank you for purchasing the OMRON R2 Wrist Blood Pressure Monitor. This remarkable, compact and easy to use instrument is ideal for people who frequently monitor their own blood pressure. The small, pre-formed wrist cuff is very convenient and easy to apply.

The OMRON R2 is a compact and easy to use blood pressure monitor, operating on the oscillometric principle. It measures your blood pressure and pulse rate simply and quickly. For comfortable controlled inflation without the need of pressure pre-setting or re-inflation the device uses its advanced "IntelliSense" technology.

The unit also stores up to 30 measurement results in memory.

Intended Use

This product is designed to measure the blood pressure and pulse rate of people within the range of the designated wrist cuff, following the instructions in this instruction manual. It is mainly designed for general household use. Please read the Important Safety Information in this instruction manual before using the unit.

Please read this instruction manual thoroughly before using the unit. For specific information about your own blood pressure, CONSULT YOUR DOCTOR.

Important Safety Information

Consult your doctor prior to using in pregnancy or if diagnosed with arrhythmia or arteriosclerosis.

Please read this section carefully before using the unit.

Warning:

Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

General Usage

Always consult your doctor. Self-diagnosis of measurement results and self-treatment are dangerous.

People with severe blood flow problems, or blood disorders, should consult a doctor before using the unit, as cuff inflation can cause internal bleeding.

Battery Usage

If battery fluid should get in your eyes, immediately rinse with plenty of clean water. Consult a doctor immediately.

Caution:

Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient or damage to the equipment or other property.

General Usage

Do not leave the unit unattended with infants or persons who cannot express their consent.

Do not use the unit for any purpose other than measuring blood pressure.

Do not disassemble the unit or wrist cuff.

Do not inflate the wrist cuff over 299 mmHg.

Do not use a mobile phone or other devices that emit electromagnetic fields, near the unit. This may result in incorrect operation of the unit.

Do not operate unit in a moving vehicle (car, airplane).

Battery Usage

If battery fluid should get on your skin or clothing, immediately rinse with plenty of clean water.

Use only two "AAA" alkaline batteries with this unit. Do not use other types of batteries.

Do not insert the batteries with their polarities incorrectly aligned.

Replace old batteries with new ones immediately. Replace both batteries at the same time.

Remove the batteries if the unit will not be used for three months or more.

Do not use new and used batteries together.

General Precautions

Do not apply strong shocks and vibrations to or drop the unit.

Do not take measurements after bathing, drinking alcohol, smoking, exercising or eating.

Do not inflate the wrist cuff when it is not wrapped around your wrist.


Do not wash the wrist cuff or immerse it in water.

Read and follow the "Important information regarding Electro Magnetic Compatibility (EMC)" in the Technical Data Section.

Read and follow the "Correct Disposal of This Product" in the Technical Data Section when disposing of the device and any used accessories or optional parts.


1. Overview

Main unit: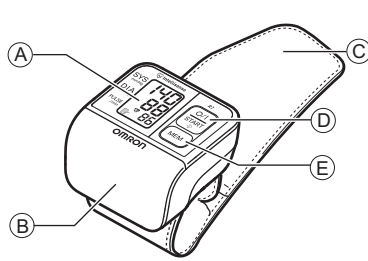

- A. Display
- B. Battery compartment cover
- C. Wrist cuff
- D. O/I START button
- E. MEM (Memory) button

Display: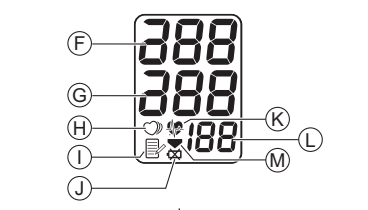

- F. Systolic blood pressure
- G. Diastolic blood pressure
- H. Irregular heartbeat symbol
- I. Memory symbol (Displayed when viewing values stored in memory)
- J. Battery low symbol
- K. Heartbeat symbol
- L. Pulse display and Memory number
- M. Deflation symbol

*Note: If your systolic or diastolic pressure is outside the standard range (above 135/85 mmHg) the Heartbeat symbol (H) will blink. Please refer to "3.3 Taking a Reading".

2. Preparation

2.1 Installing/Replacing the Batteries

Remove the battery compartment cover by pulling it off in the direction of the arrow.

1) Push the hook on the bottom of the battery cover.

2) Pull the cover off the main unit.

3) Put the battery cover back in place.

Note: The measurement values continue to be stored in memory even after the batteries are replaced.

2.2 Battery Life & Replacement

If the battery low symbol (L) appears on the display, replace both batteries at the same time.

- When the battery low symbol (L) starts to blink, you will still be able to use the unit for a short while. You should replace the batteries with new ones as soon as possible.

- When the symbol (L) remains lit, the batteries are exhausted. You should replace the batteries with new ones at once.

Turn the unit off before replacing the batteries.

Remove the batteries if the unit will not be used for three months or more.

Dispose of batteries according to applicable local regulations.

Two new identical 1.5V "AAA" alkaline batteries will last for approximately 300 measurements, when used to take two measurements a day.

Since the supplied batteries are for monitoring use only, they may have a shorter life and do not last for 300 measurements.

3. Using the Unit

3.1 Applying the Wrist Cuff

You can take a measurement on either your left or right wrist.


Notes: You can take a measurement on either your left or right arm.

The blood pressure can differ between the right arm and the left arm and therefore also the measured blood pressure values can be different. Omron recommends to always use the same arm for measurement. If the values between the two arms differ substantially, please check with your physician which arm to use for your measurement.

To ensure correct measurement, apply the wrist cuff so that it fits comfortably around your wrist.

Do not apply over clothing.

1. Place the wrist cuff over your left wrist with your left thumb facing upward.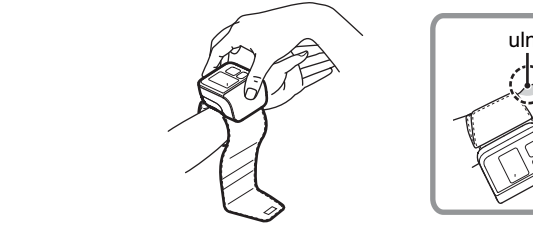

Note: Make sure that the wrist cuff does not cover the protruding part of the wrist bone (ulna) on the outside of the wrist.

2. Hold the bottom part of the wrist cuff and wrap it around the wrist while pulling so that it fits comfortably.

Notes: Unless the wrist cuff is wrapped securely around the wrist, it will not be possible to take correct measurements. If you have a slender wrist, the cuff may feel slightly loose. However, this will not affect measurement results and you should not force the cuff to fit.

The remaining part of the wrist cuff can be conveniently folded back out of the way.

If the remaining part of the cuff is difficult to fold back, measurement results will not be effected.



Taking a reading on the right wrist

Measurements can also be made on the right wrist. Fit the unit on the right wrist as shown.

3.2 Correct Posture


Correct posture during measurement helps you get accurate results. You should also try to measure your blood pressure at the same time each day.

Notes: Do not take measurement after bathing, drinking alcohol, or exercising. Do not move or talk during measurement.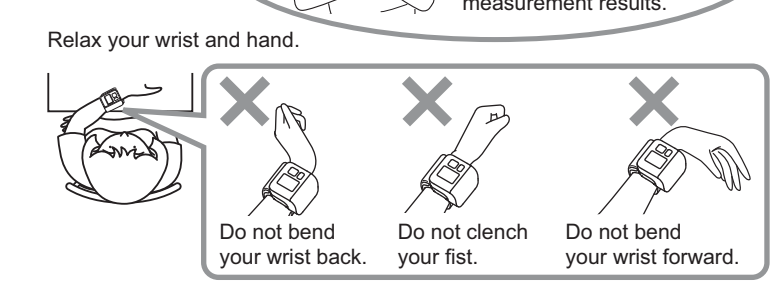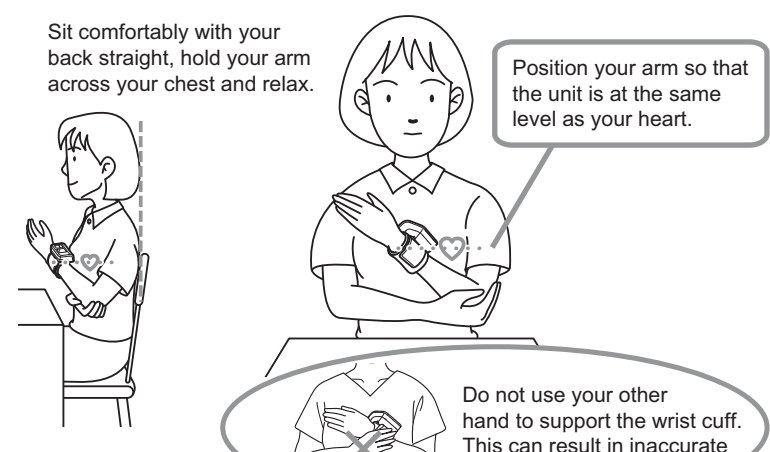

3.3 Taking a Reading

1. Press the O/I START button to start measurement. The wrist cuff will start to inflate automatically after the power is turned on.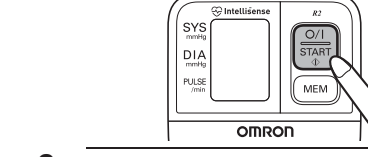

2. After the unit has detected your blood pressure and pulse rate, the cuff automatically deflates and your blood pressure and pulse rate are displayed.

Notes: Sit still and do not talk or move until the measurement is completed. Keep the unit at heart height until the measurement is completed. To stop measurement, press the O/I START button at any time during measurement.

3. Check the measurement result. The unit automatically stores blood pressure and pulse rate into its memory. Refer to "3.4 Using the Memory Function".

4. Undo the cuff and remove the unit.

5. Press the O/I START button to turn off the unit. If you forget to turn off the unit, it will shut itself off automatically after two minutes.

Important: If your systolic or diastolic pressure is outside the standard range, the heartbeat symbol will blink when the measurement result is displayed. Recent research suggests that the following values can be used as a guide to high blood pressure for measurements taken at home.

Systolic Blood Pressure	Above 135 mmHg
Diastolic Blood Pressure	Above 85 mmHg

This criteria is for home blood pressure measurement. Your blood pressure monitor includes an irregular heartbeat feature. Irregular heartbeats can influence the results of the measurement. The irregular heartbeat algorithm automatically determines if the measurement is usable or needs to be repeated. If the measurement results are affected by irregular heartbeats but the result is valid, the result is shown together with the irregular heartbeat symbol (H). If the irregular heartbeats cause the measurement to be invalid, no result is shown. If the irregular heartbeat symbol (H) is shown after you have taken a measurement, repeat the measurement. If the irregular heartbeat symbol (H) is shown frequently, please make your doctor aware of it.


3.4 Using the Memory Function

This unit has a memory capable of storing 30 sets of readings. Every time you complete the measurement, the unit automatically stores blood pressure and pulse rate.

Note: When 30 sets of readings are stored in memory, the oldest set will be deleted to store a new set.

To View Previous Readings Stored in Memory

1. Press the MEM button to view readings stored in memory from the most recent to the oldest. The Memory number appears for a second before the pulse rate is displayed. The newest set is numbered "1".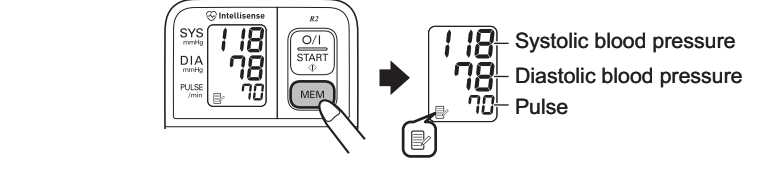


Press the MEM button repeatedly to cycle through the readings.

2. Press the O/I START button to turn the unit off. Note: If you forget to turn the unit off, it will automatically shut itself off after two minutes.

To Delete All Values in Memory

You cannot delete the stored readings partially, all the readings in the unit will be deleted. To delete stored readings, first press the MEM button. Then while holding it down, press the O/I START button simultaneously for about 2-3 seconds. All readings will then be deleted.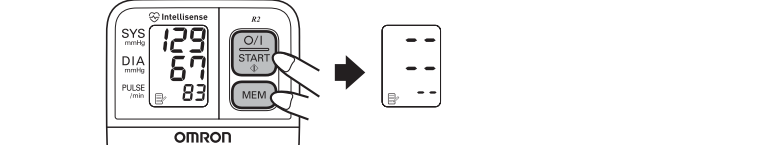

Note: After the reading have been deleted, the screen to the right of the illustration above appears and Memory symbol flashes. Release the buttons to return to the memory display.

4. Handling Errors and Problems

4.1 Error Messages

Error Symbol	Cause	Correction
Heartbeat symbol (H)	Irregular or weak pulses are detected.	Remove the wrist cuff. Wait 2-3 minutes and then take another measurement. Repeat the steps in section 3.1. If this error continues to appear, contact your doctor.
Cuff over inflated symbol (C)	Cuff over inflated.	Repeat measurement. Remain still and do not move during measurement. (Refer to "3.3 Taking a Reading".)
Movement during measurement symbol (M)	Movement during measurement.	Carefully read and repeat the steps listed under "3.1 Applying the Wrist Cuff".
Hardware failure symbol (E)	An Er mark with a code/number indicates the device has a hardware failure.	Consult your OMRON retail outlet or distributor.
Battery low symbol (L)	The battery power is low.	Replace the batteries with two new "AAA" alkaline (LR03) batteries.

Note: The irregular heartbeat symbol (H) may also be displayed with error messages.


4.2 Troubleshooting

Symptom	Cause	Correction
The reading is extremely low (or high).	Are you holding the wrist cuff at heart level?	Measure while in the correct posture. (Refer to "3.2 Correct Posture".)
	Is the cuff wrapped snugly around the wrist?	Wrap the cuff correctly. (Refer to "3.1 Applying the Wrist Cuff".)
	Are your arms and shoulders tense?	Relax and try taking the measurement again. (Refer to "3.3 Taking a Reading".)
Wrist cuff pressure does not rise.	Movement or talking during measurement.	Remain still and do not talk during measurement. (Refer to "3.3 Taking a Reading".)
	Air is leaking from the wrist cuff.	Consult your OMRON retail outlet or distributor.
Wrist cuff deflates too soon.	The wrist cuff is loose.	Apply the cuff correctly so that it is securely wrapped around the wrist. (Refer to "3.1 Applying the Wrist Cuff".)
The blood pressure is different each time. The reading is extremely low (or high).		Blood pressure readings constantly vary with time of day and how relaxed you are. Take several deep breaths and try to remain relaxed before taking a measurement.
The unit loses power during measurement.	The batteries are drained.	Replace the batteries with new ones.
Nothing happens when you press the buttons.	The batteries are drained.	Replace the batteries with new ones.
	The batteries have been inserted incorrectly.	Insert the batteries with the correct (+/-) polarity.
Other problems.	Press the O/I START button and repeat measurement.	
	If the problem continues, try replacing the batteries with new ones. If this still does not solve the problem, contact your OMRON retail outlet or distributor.	

5. Maintenance and Storage

To protect your unit from damage, please avoid the following:

- Subjecting your unit to extreme temperatures, humidity, or direct sunlight.
- Washing the cuff or exposing the cuff or unit to water.
- Disassembling the unit.
- Subjecting the unit to strong shocks or vibrations. Dropping the unit.
- Cleaning the unit with volatile liquids.

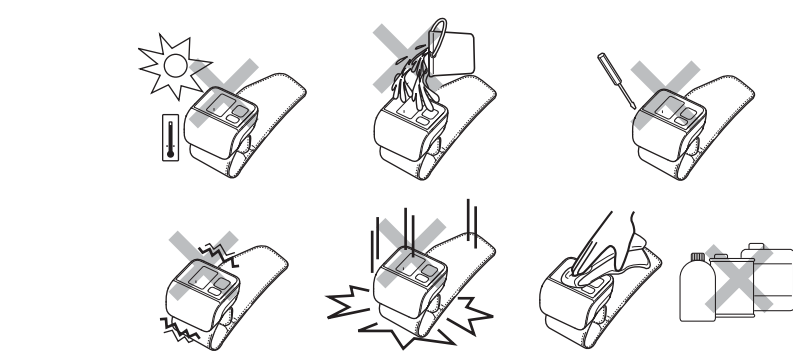- The unit should be cleaned with a soft, dry cloth.
- Use a soft, moistened cloth and soap to clean the cuff.
- Keep the unit in its storage case when not in use.
- Fold the cuff into the storage case.

Do not store the unit in the following situations:

- If the unit is wet.
- Locations exposed to extreme temperatures, humidity, direct sunlight, dust or corrosive vapours.
- Locations exposed to vibrations, shocks or where it will be at risk of falling.

Calibration and Service

The accuracy of this blood pressure monitor has been carefully tested and is designed for a long service life. It is generally recommended to have the unit inspected every two years to ensure correct functioning and accuracy. Please consult your authorised OMRON dealer or the OMRON Customer Service at the address given on the packaging or attached literature. If the wrist cuff needs to be replaced have this done by an authorised expert. Consult your authorised OMRON dealer or the OMRON Customer Service. Do not carry out any repairs by yourself. If a defect occurs or you have doubts about the correct functioning of the device, consult your authorised OMRON dealer or the OMRON Customer Service.

6. Technical Data

Product Description	Wrist Blood Pressure Monitor
Model	OMRON R2 (HEM-6113-RU)
Display	LCD Digital Display
Measurement	Oscillometric method
Measurement Range	Pressure: 0 mmHg to 299 mmHg Pulse: 40 to 180 beats/min
Memory	30 Measurements
Accuracy	Pressure: within ±3 mmHg Pulse rate: within ±5% of reading
Inflation	Automatic inflation by pump
Deflation	Automatic rapid deflation
Power Source	Two 1.5V "AAA" alkaline (LR03) batteries
Battery Life	Approximately 300 measurements when used twice daily with alkaline batteries at a room temperature of 23°C

Applied Part: Type B

Operating Temperature/Humidity: 10°C to 40°C, 30 to 85% RH

Storage Temperature/Humidity: -20°C to 60°C, 10 to 95% RH

Weight of Main Unit: Approximately 117 g (not including batteries)
Outer Dimensions: 71 mm (w) x 41 mm (h) x 70 mm (d) (not including the wrist cuff)
Measurable circumference of wrist: Approximately 13.5 to 21.5 cm

Cuff material: Nylon
Cuff Package Content: Main unit, storage case, two "AAA" alkaline (LR03) batteries, instruction manual, guarantee card, blood pressure pass

Note: Subject to technical modification without prior notice.

CE 0197

- This device fulfils the provisions of EC directive 93/42/EEC (Medical Device Directive).
- This blood pressure monitor is designed according to the European Standard EN1060, Non-invasive sphygmomanometers Part 1: General Requirements and Part 3: Supplementary requirements for electromechanical blood pressure measuring systems.
- This OMRON product is produced under the strict quality system of OMRON HEALTHCARE Co. Ltd., Japan. The Core component for OMRON blood pressure monitors, which is the Pressure Sensor, is produced in Japan.
- Disposal of this product and used batteries should be carried out in accordance with the national regulations for the disposal of electronic products.

Important information regarding Electro Magnetic Compatibility (EMC)

With the increased number of electronic devices such as PC's and mobile (cellular) telephones, medical devices in use may be susceptible to electromagnetic interference from other devices. Electromagnetic interference may result in incorrect operation of the medical device and create a potentially unsafe situation. Medical devices should also not interfere with other devices.

In order to regulate the requirements for EMC (Electro Magnetic Compatibility) with the aim to prevent unsafe product situations, the EN606061-1-2:2007 standard has been implemented. This standard defines the levels of immunity to electromagnetic interferences as well as maximum levels of electromagnetic emissions for medical devices.

This medical device manufactured by OMRON HEALTHCARE conforms to this EN606061-1-2:2007 standard for both immunity and emissions. Nevertheless, special precautions need to be observed: Do not use mobile (cellular) telephones and other devices, which generate strong electrical or electromagnetic fields, near the medical device. This may result in incorrect operation of the unit and create a potentially unsafe situation. Recommendation is to keep a minimum distance of 7 m. Verify correct operation of the device in case the distance is shorter.

Further documentation in accordance with EN606061-1-2:2007 is available at OMRON HEALTHCARE EUROPE at the address mentioned in this instruction manual. Documentation is also available at www.omron-healthcare.com.

Correct Disposal of This Product (Waste Electrical & Electronic Equipment)

This marking shown on the product or its literature, indicates that it should not be disposed of, with other household wastes at the end of its working life. To prevent possible harm to the environment or human health from uncontrolled waste disposal, please separate this from other types of wastes and recycle it responsibly to promote the sustainable reuse of material resources.

Household users should contact either the retailer where they purchased this product, or their local government office, for details of where and how they can take this item for environmentally safe recycling.

Business users should contact their supplier and check the terms and conditions of the purchase contract. This product should not be mixed with other commercial wastes for disposal.

This product does not contain any hazardous substances.

Manufacturer	OMRON HEALTHCARE Co., Ltd. 53, Kunotsubo, Terado-cho, Muko, Kyoto, 617-0002 JAPAN
EU-representative	OMRON HEALTHCARE EUROPE B.V. Scorpius 33, 2132 LR Hoofddorp, THE NETHERLANDS www.omron-healthcare.com
Exclusive distributor in Russia & importer	CompletService Ltd. 13-14 building, 26 B, Tishinskiy per. Moscow 123557, RUSSIA www.csmedica.ru
Production Facility	OMRON (DALIAN) CO., LTD. DALIAN, CHINA

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