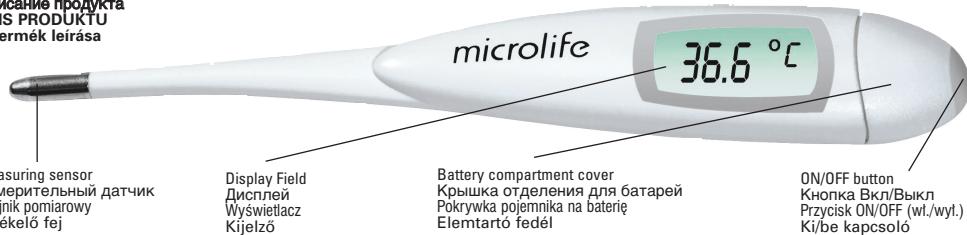
DIGITAL JUMBO LCD THERMOMETER

MT 1951

DESCRIPTION OF THIS PRODUCT

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20 seconds *
20 секунд *
20 sekund *
20 másodpercben *

35.2 °C – 36.7 °C

* 90 seconds after beep tone (depending on ambient temperature)

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ELEMCSERE

When the ▼-symbol appears, battery is exhausted and needs replacing. Please remove the compartment lid of the thermometers and replace the exhausted battery by positioning the + at the top.

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CE 0044

IMPORTANT SAFETY INSTRUCTIONS

- The thermometer is suitable exclusively for measurement of the body temperature!
- The minimum measurement time until the signalling tone [beep] must be maintained without exception!
- The thermometer contains small parts (battery, etc.) that can be swallowed by children. Therefore never leave the thermometer unattended to children.
- Protect the thermometer from hitting and striking!
- Avoid ambient temperatures above 60°C. NEVER boil the thermometer!
- Use only the commercial disinfectants listed in the section «cleaning and disinfecting» to clean the thermometer. The thermometer must be intact when immersed in liquid disinfectant.
- Warning:** The thermometer contains a long-life battery. For disposal, please comply with local regulations for special waste.
- The manufacturer recommends to verify the accuracy by an authorised laboratory every 2 years.

TURNING ON THE THERMOMETER

To turn on the thermometer, push the button on the top; a short signalling tone [beep] signals «thermometer ON». A display test is performed. Then at an ambient temperature of less than 32°C, an « \leftarrow » and a flashing « $^{\circ}\text{C}$ » appear at the upper right of the display. The thermometer is now ready for use.

USING THE THERMOMETER

During a measurement, the current temperature is displayed continuously. The « $^{\circ}\text{C}$ » symbol flashes during this period of time. If the signalling tone sounds [beep-beep-beep-beep 10 times] and the « $^{\circ}\text{C}$ » is no longer flashing, this means that the measured increase in temperature is now less than 0.1 °C in 8 seconds and that the thermometer is ready for reading. To prolong the life of the battery, turn it off after use by pressing the operating button briefly. Otherwise the thermometer will automatically turn off after about 10 minutes.

STORAGE OF MEASURED VALUES

If you hold the button down for more than 3 seconds when turning the thermometer on, the maximum temperature stored automatically at the last measurement will be displayed. At the same time, a small «M» for memory will appear at the right of the display. After 3 more seconds, this value will disappear and the thermometer will switch to the normal measurement mode. Then the stored value is erased.

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ILLUMINATED DISPLAY / NITE GLOW FEATURE

The thermometer will beep and the Nite Glow will come ON simultaneously for 5 seconds when you push the on-off button to turn ON the unit, the display will read "188.8" for the first 2 seconds then automatically display the last temperature for the other 3 seconds. When the power button is pressed to Switch-Off the unit, there will be a 4-second additional delay with the Nite Glow ON before power off, which allows user to read the measurement reading easily in the dark.

WARRANTY

We give you a lifetime warranty after the date of purchase. Any damage caused by improper handling shall not be covered by the warranty. Batteries and packaging are also excluded from the warranty. All other damage claims excluded. A warranty claim must be submitted within the warranty period. Please package your defective thermometer well and send with sufficient postage to:

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